

SMICHAE

www.sowonbio.co.kr

PN2

sowonbio

SMC HANE

- 01** PRODUCT INTRODUCTION
- 02** KEY INGREDIENTS & BENEFITS
- 03** DIRECTIONS FOR USE & DOSAGE
- 04** RECOMMENDED TREATMENT INTERVAL
- 05** PRECAUTIONS BEFORE & AFTER USE

1.PRODUCT INTRODUCTION



Category	Details
Product Name	SMICHAE PN 2
Packaging Unit	2.5ML /1BOX
Storage Conditions	Store at room temperature (2-25°C), away from direct sunlight.
Key Ingredients & Content	Polynucleotide2%,Sodium Hyaluronate1%
Treatment Area	Face, back of hands, neck, etc.

2.INGREDIENTS & BENEFITS

PN 2%

HA 1%

Recommended for

- Those experiencing **severe inner dryness**
- Those with a **compromised skin barrier**
- Those with **rough and dull skin**
- Those concerned about **fine lines, enlarged pores, or sensitive skin**

• Benefits of PN

Improve skin elasticity
Reduce appearance of pores
Reduce sebum and dead skin cells
Improve skin texture and tone
Restore oil & moisture balance
Regenerate and strengthen the skin barrier
Improve appearance of fine lines

• Benefits of HA

Hydration
Prevention of Skin Damage
Skin Regeneration

3.DIRECTIONS FOR USE & DOSAGE

1. Anesthetize and disinfect the treatment area

Before the procedure, disinfect the treatment area and apply a topical anesthetic cream.

2.Needle Selection

Manual Injection: **1.5 mm** needle, **4 mm** needle (painless)

Injector: Select the appropriate needle according to the practitioner's and patient's condition.

3.Administration Method & Treatment Interval

Manual Injection

Injection depth: 1.0–4.0 mm

Recommended dosage: Inject 0.05–0.25 mL per point, with injection points spaced 0.5–2 cm apart.

Recommended amount: 1–2 syringes per treatment session.

Injector

Recommended setting: 0.05 mL per point.

When mixing with other products, for a total volume of 5–10 mL, it is recommended to set the injector to 0.1 mL per point.


4.Recommended Dosage

SMICHAЕ PN 2 is a combination of PN + HA and has a viscous consistency. It is recommended to use 1 syringe (2.5 mL) per treatment session. For treatment including the neck area, a total volume of up to 5 mL is recommended. It can also be used in combination with other products.

*Note: The administration method may be adjusted according to the practitioner's expertise and technique.

4.TREATMENT INTERVAL

RECOMMENDED TREATMENT INTERVAL

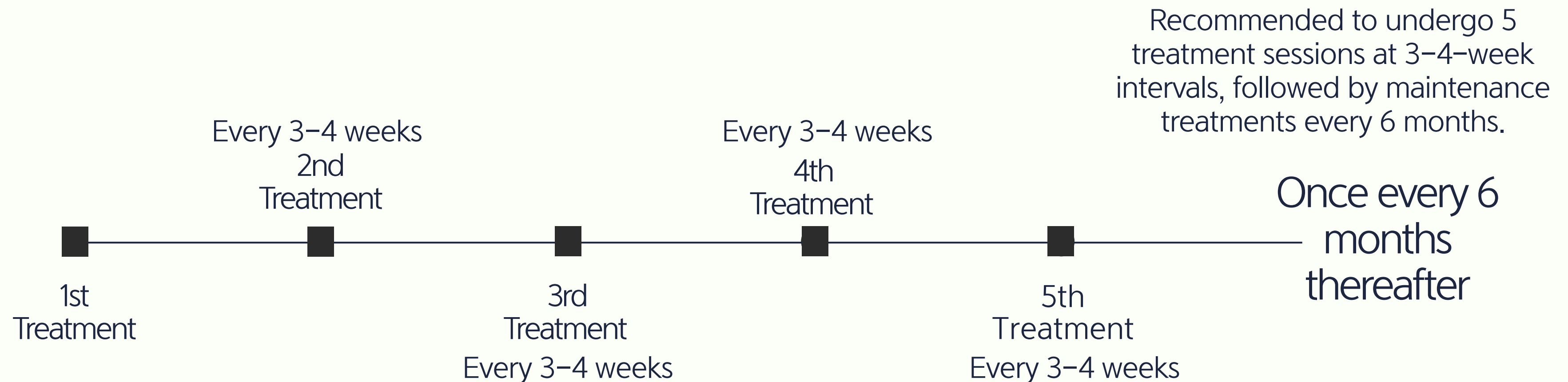

*Note: The administration method may be adjusted according to the practitioner's expertise and technique.

5.PRECAUTIONS BEFORE & AFTER USE

Pre-Treatment Checklist & Precautions

History of Allergies	Any allergies to salmon or fish
Pregnancy or Breastfeeding Status	Possibility of Pregnancy or Current Breastfeeding Status
Skin Condition	If inflammation, infection, or irritation is present, postpone treatment.
Pain Sensitivity	Pain concerns, anxiety, and preference for anesthesia.

Post-Treatment Precautions

On the Day of Treatment	Avoid excessive irritation during cleansing or makeup application.
1 Week After Treatment	Avoid strenuous exercise, alcohol, saunas, strong massage, rubbing, exfoliation, scrubs, and peels.
Sun Protection	Moisturizing and sun protection are essential.
Check for Side Effects	Worsening or persistent swelling/bruising requires medical attention; temporary bumps may occur.

Thank You

sowonbio

customercenter@sowonbio.kr