


4UCOLLAFEEL

Highly purified bio collagen ,the secret of
healthy and supple skin

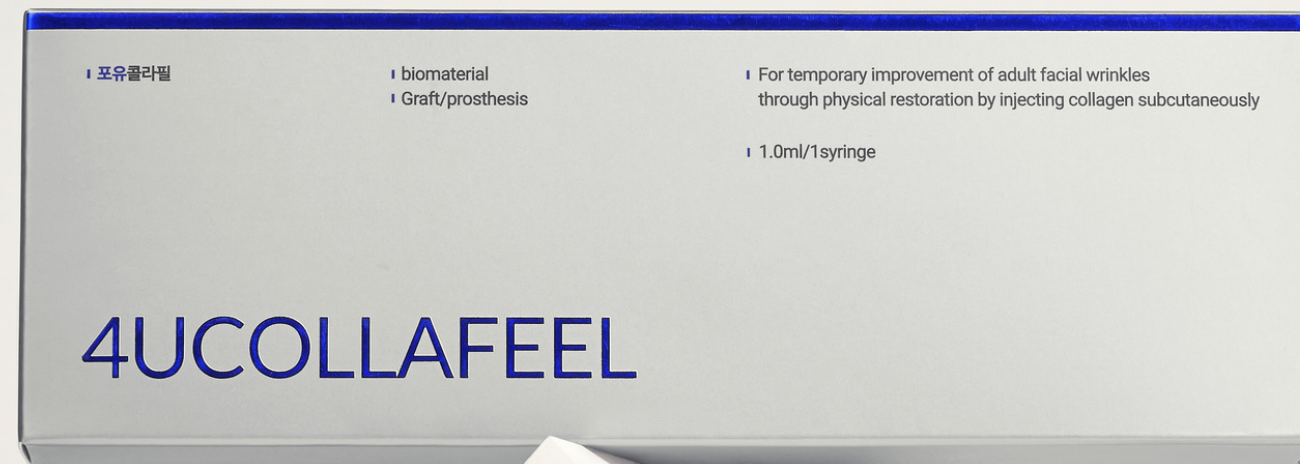

WHY IS COLLAGEN SO IMPORTANT?

Almost 100% of the population of South Korea and Japan are aware of the existence of collagen.

According to surveys, in these countries, 90% of people over 60 years old and 83% of 20-year-olds know about collagen.

Collagen plays an important role in preserving the youth and beauty of the skin. Protein is responsible for its smoothness, elasticity, strength and elasticity.

It stabilizes the oval of the face and evens out its tone.

It is also responsible for the regeneration of tissues after their damage, accelerating the healing process.

Collagen is the framework, the "building material" of all connective tissues in our body.

It is responsible for the work of joints, ligaments, muscles , teeth and bones, and for the quality of the skin.

The body produces collagen itself, but its synthesis decreases with age.



WHAT IS BIO COLLAGEN?

4UCOLLAFEEL is a natural type I collagen.

The product restores skin turgor of the face, neck, décolleté and hands, body, slows down aging. It gives a thickening of the skin, without additional volume.

The common existing components of collagen (gelatin, collagen peptides) are antigens to the human body.

They have a "telopeptide" , which can act as (a substance considered foreign in living organisms).

Thanks to enzyme technology, the natural structure of the protein is not altered and does not involve the chemical formation of cross-links.

The safety and absolute hypoallergenicity of the product is achieved by separating the triple collagen helix from the telopeptide end amino acids.

The collagen fiber retains its natural triple helix structure, molecular weight and molecular chain sequence, while eliminating any possibility of allergies and complications.



4UCOLLAFEEL



No side effects due to the removed telopeptide, which is an antigen.



Pure collagen that does not undergo denaturation



**Korean Food and Drug
Administration**

The times when collagen preparations gave immune reactions are long gone!

4UCOLLAFEEL is a high-tech preparation of biological origin, which structure is as close as possible to the collagen produced by our body.

Bio collagen not only quickly and effectively restores damaged collagen chains in the skin, but also restores cellular activity.

There is an improvement in skin hydration due to the use of micronized particles and restoration of cell membrane charges. Collagen skin therapy is a course treatment based on based on the sequential stimulation of fibroblasts in order to obtain long-term results.

Clinical results of randomized multifunctional studies reflect a high degree of tolerability and effectiveness of therapy with injectable collagen preparation.

4UCOLLAFEEL



A COURSE OF THREE
PROCEDURES IS ENOUGH



DOES NOT REQUIRE
REHABILITATION



THE EFFECT LASTS UP TO ONE
AND A HALF YEARS



INDICATIONS


- REGENERATION OF DERMAL AND SUBDERMAL CONNECTIVE TISSUE BY STIMULATING FIBROBLASTS;
- SKIN TONING;
- REDUCING WRINKLES IN THE CORRECTION AREA;
- THE RETURN OF LOST VOLUMES;
- HYPOTENSION OF THE SKIN IN VARIOUS AREAS OF THE FACE, NECK, DECOLLETE AND BODY (AGE-RELATED, ARISING AFTER PREGNANCY, LACTATION, SUDDEN JUMPS IN BODY WEIGHT, AN UNBALANCED DIET).




- SCARS ON THE SKIN OF THE FACE AND BODY (ATROPHIC, POST-TRAUMATIC, POST-SURGICAL, POST-ACNE);
- IMPROVING THE QUALITY OF THE SKIN;
- RESTORING THE NATURAL OVAL OF THE FACE;
- AGE-RELATED SKIN CHANGES (DERMAL CREASES, ELASTOSIS);
- REGENERATION OF DERMAL AND SUBDERMAL CONNECTIVE TISSUE
REGENERATION OF DERMAL AND SUBDERMAL CONNECTIVE TISSUE;

PROTOCOL


It is introduced using one of the techniques
(injection only)

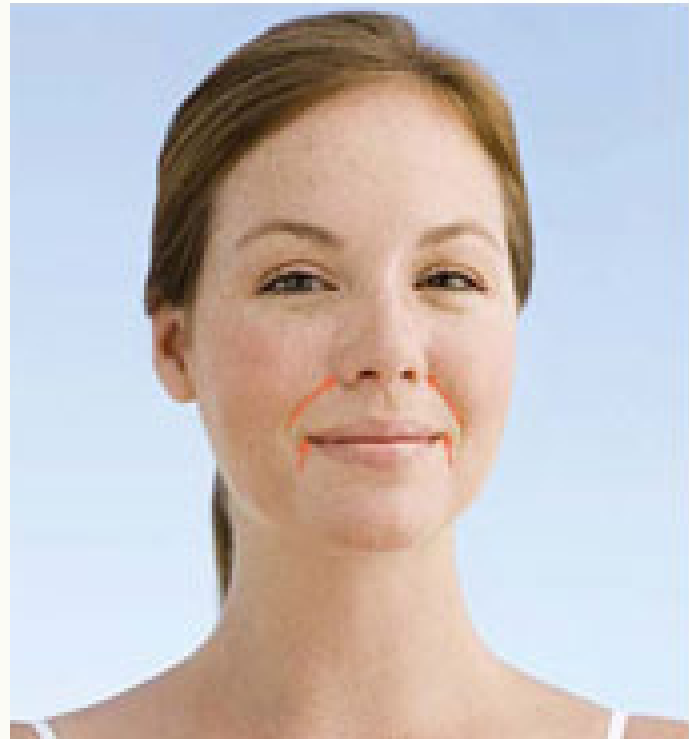

Linear technique

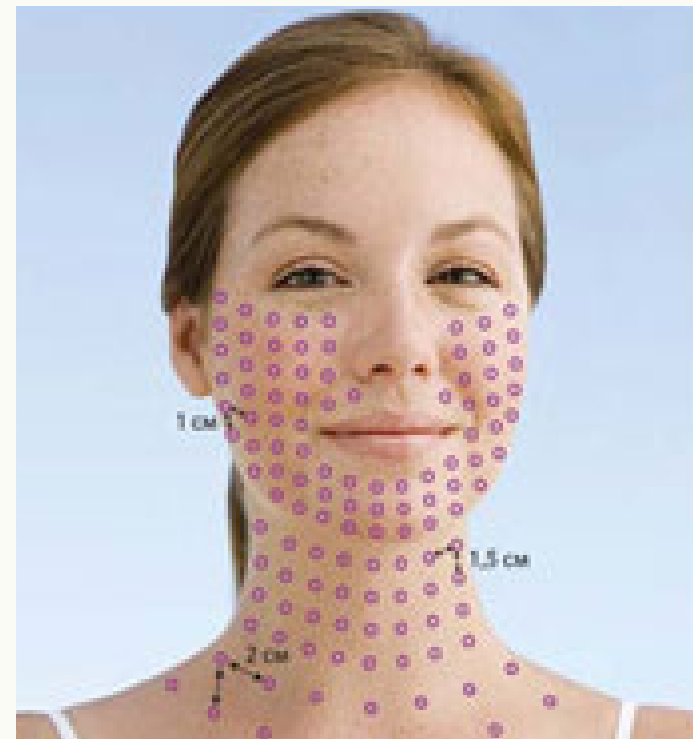

Papular technique


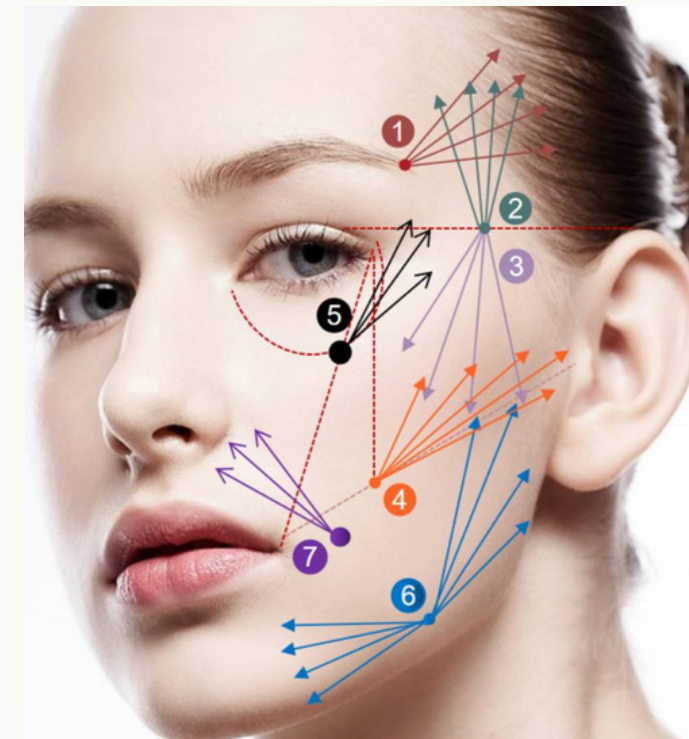


Vector lifting



PRP therapy

LINEAR TECHNIQUE

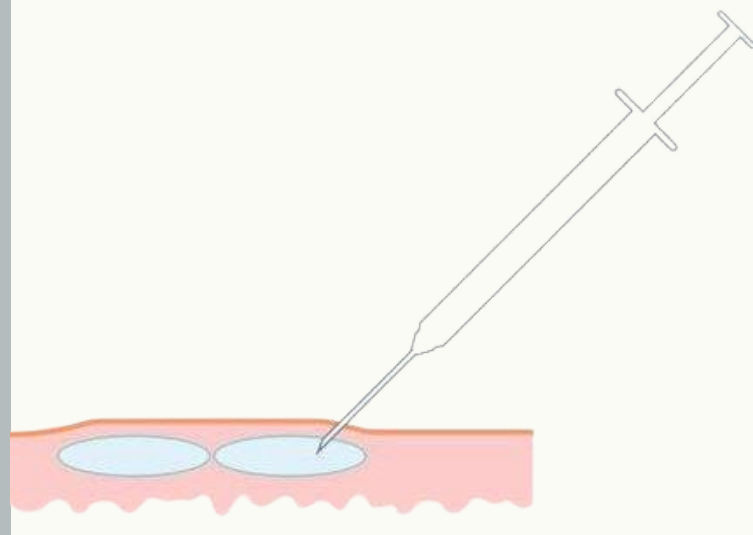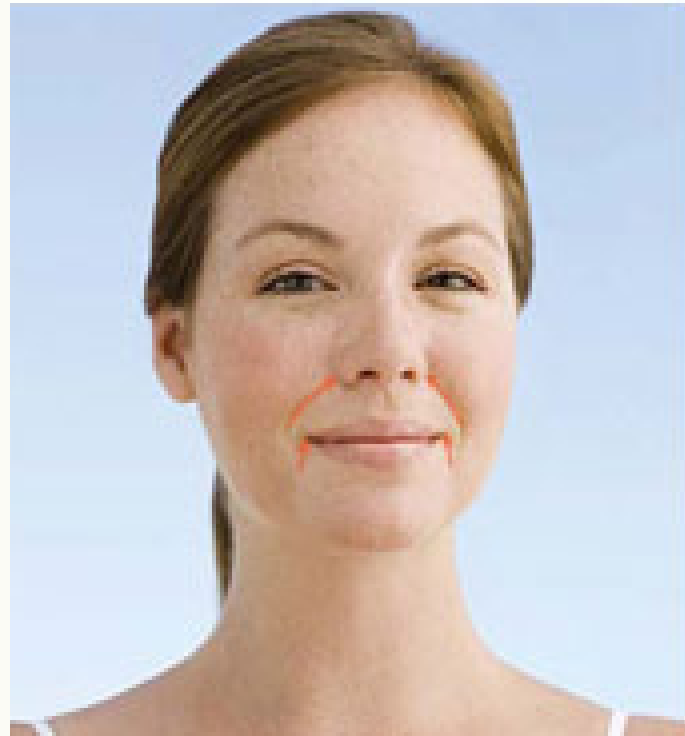

The technique involves the formation of papules of various sizes in order to improve the deposition of medicinal substances.

Papules vary in diameter, as well as in the depth of injection of the drug.

The advantage of the papular technique is the maximum prolongation of the therapeutic effect of the drugs.

Superficial papules, micropapules.



Superficial papules are produced in areas with poorly defined subcutaneous fat (frontal, paraorbital zygomatic areas, angles of the lower jaw).

The depth of introduction of the cocktail is 1-2 mm, the angle of the needle is 10-15 degrees. The needle is held with the cut side up.

The diameter of the papules should not exceed 1.5-2 mm.

Using this technique, you can introduce any cocktails. Hydrated in 5 ml of saline solution, 1: 1 ml of the finished solution is used in a fine-papular technique in the area of the upper, lower eyelid and outer corner of the eye to correct static wrinkles, increase skin density and tone, reduce translucency of the underlying anatomical structures under the dermis

PAPULAR TECHNIQUE

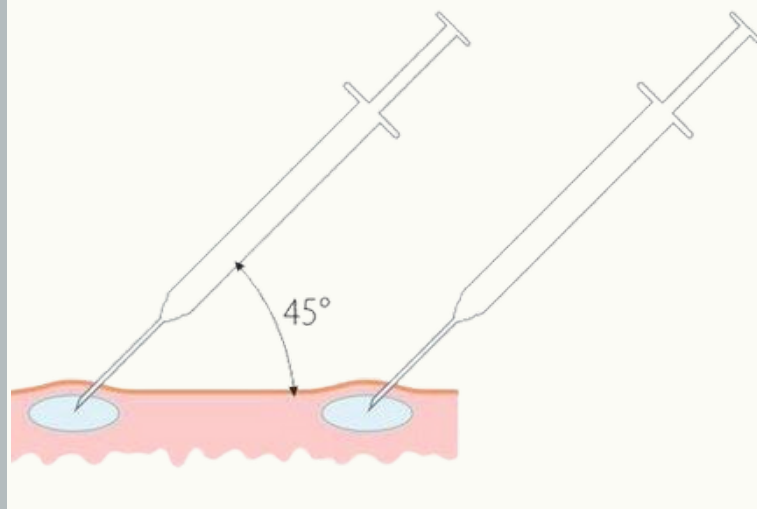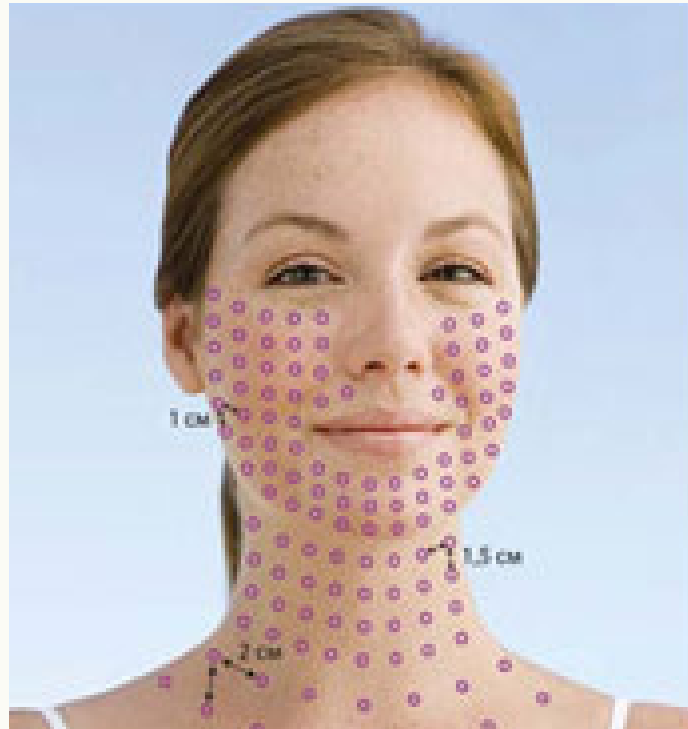

The technique consists in forming a linear channel in the dermis and filling it evenly on the return stroke of the needle. This technique is used in the treatment of wrinkles, striae, and scars. In this way, preparations of gel consistency are mainly administered.


There are many varieties of linear technology that differ in the configuration of individual channels:

- tracer;
- tunnel;
- the fan;
- reinforcement technique ("mesh").

The procedure can be performed every 2-3 weeks, depending on the severity of aesthetic problems. The course consists of 3-5 procedures, after which you can take a break and return to the course in a year or offer the patient supportive procedures with an interval of 3-4 months.

It is hydrated in 5 ml of saline solution, 1: 1 ml of the finished solution is used to correct static wrinkles, increase skin density and tone, and reduce the translucency of the underlying anatomical structures under the dermis.

VECTOR LIFTING

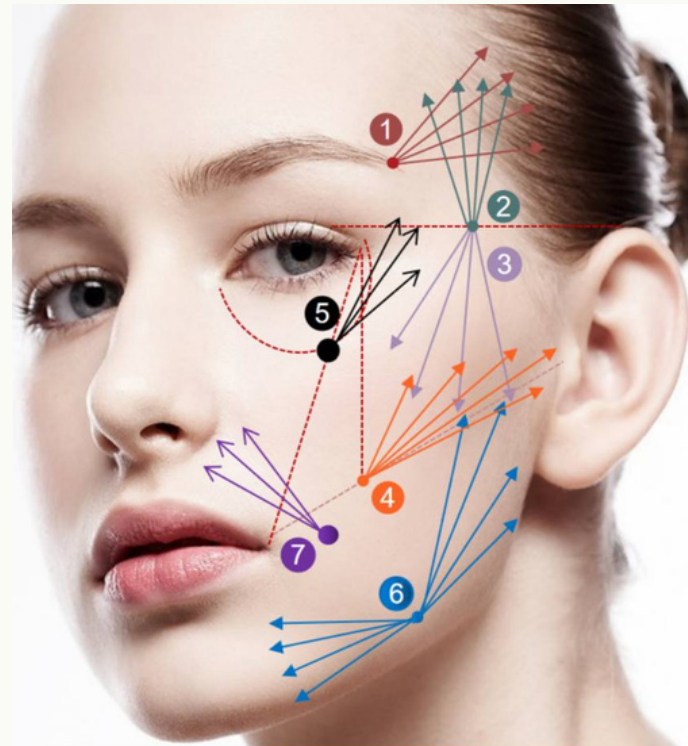

Consumption of the preparation per one zone per procedure:

- | | |
|-----------|-----------------|
| 1 . 0.5ml | 4 . 0.5ml |
| 2 . 0.5ml | 5 . 0.2ml |
| 3 . 0.5ml | 6 . 0.3 - 0.5ml |
| | 7 . 0.5ml |

The technique consists of creating a mechanical frame that is filled with a filler thread. Isosceles diamonds with a side length of 12 mm are drawn on the skin (which corresponds to the length of the filler syringe needle).

Using the linear technique, injections are made intradermally along the entire length of the needle along the intended sides of the diamonds. This is accompanied by rocking movements of the needle from side to side to obtain maximum compression in the tissue.

This technique is used in the treatment of wrinkles, oval shape, strengthening skin turgor.

The procedure is performed with a 25G50 cannula

Hydrated in 5 ml of saline solution, 4: 1 ml is mixed with 1-2 ml of a cocktail based on certified native HA and injected into all static wrinkles and dehydrated areas of the face.

The procedure can be performed once every 2-3 weeks, depending on the severity of aesthetic problems.

The course consists of 3–5 procedures, after which you can take a break and return to the course in a year, or offer the patient maintenance procedures at intervals of 3–4 months.

PRP therapy



PRP therapy

The use of platelet-rich plasma (PRP) for skin rejuvenation, reduces wrinkles and defects, among other things. acne scars. Platelet-rich plasma (abbreviated PRP) is the blood plasma of a patient with an increased platelet count. Such platelets contain growth factors that stimulate the healing process. It is hydrated in 5 ml of saline solution, 2: 1 ml is mixed with the patient's blood plasma in the required amount, depending on the area of the scar elements, the drug is administered popularly with hypercorrection.

The procedure can be performed every 2-3 weeks, depending on the severity of aesthetic problems. The course consists of 3-5 procedures, after which you can take a break and return to the course in a year or offer the patient supportive procedures with an interval of 3-4 months.

RESULTS BEFORE & AFTER

BRIGHTENING , NASOLABIAL FOLDS



BEFORE



AFTER

POST-ACNE SCARRING



BEFORE



AFTER

TONE, SKIN RADIANCE, NARROWING PORES



BEFORE




AFTER

DECOLTE ZONE





BEFORE

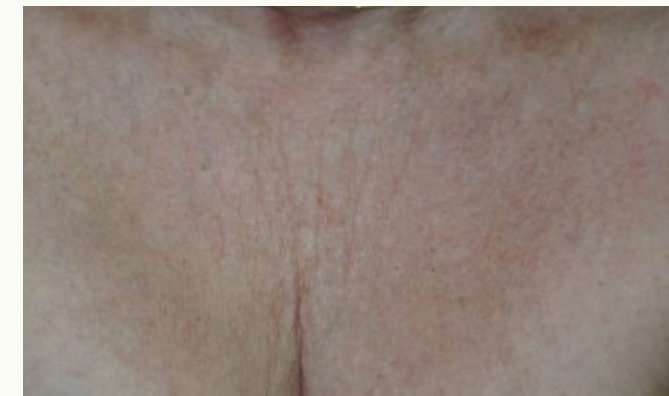

AFTER

Collagen therapy 4UCOLLAFEEL with a double effect in the complex

- with PRP therapy
- with ROYAL EYE filler
- with SMICHAE FERILO biorevitalizant

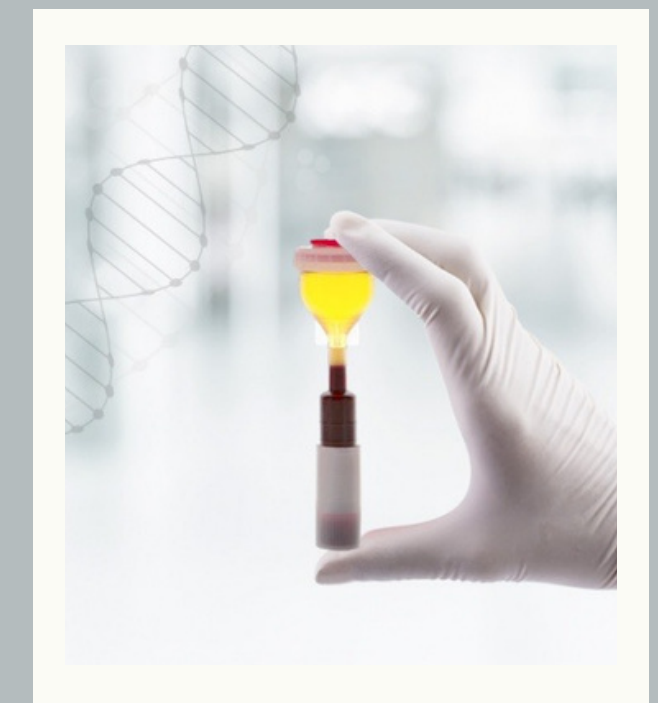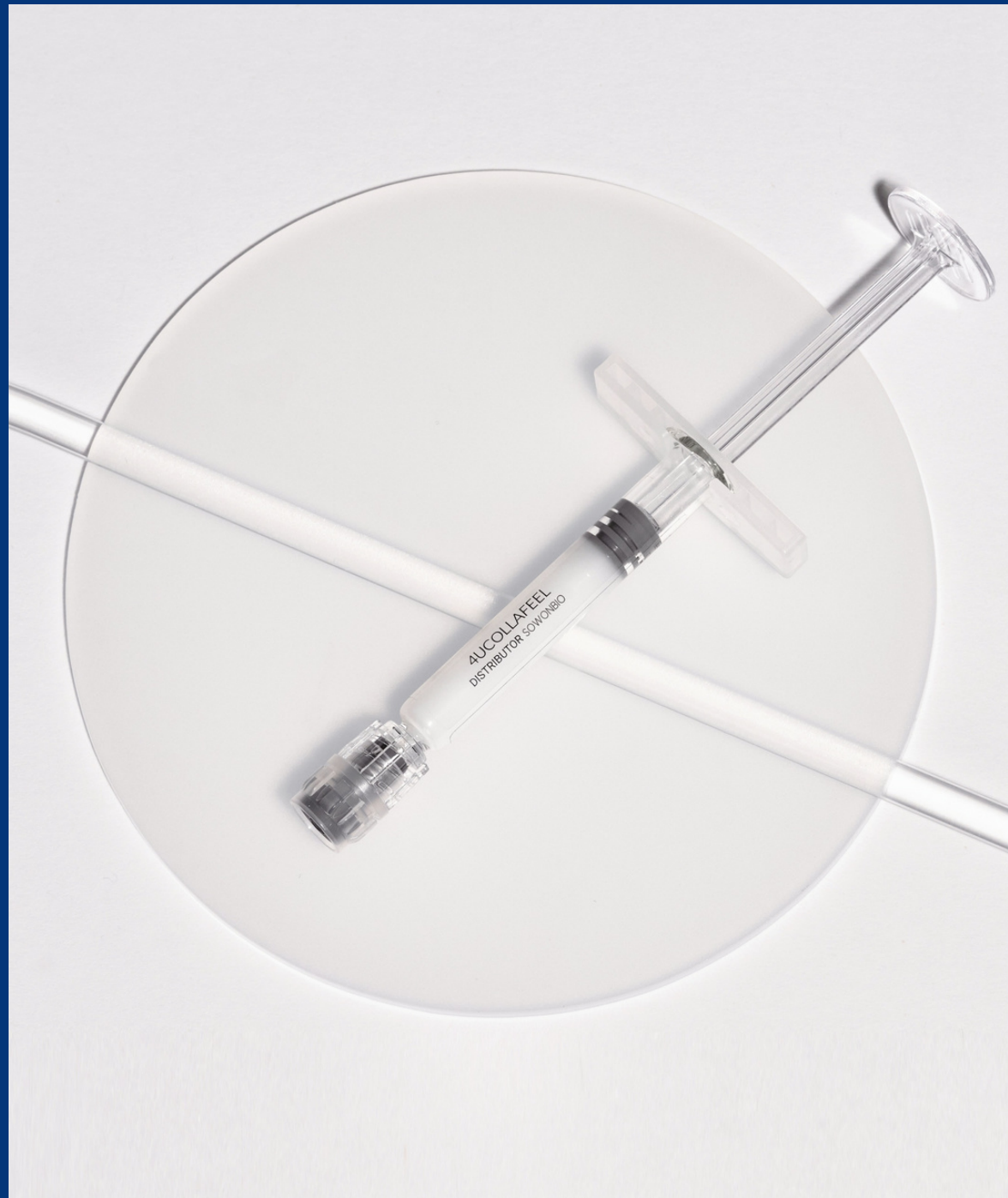

4UCOLLAFEEL

Youth and beauty in one syringe

Name	4UCOLLAFEEL
Package	1 syringe / 1ml
Main Ingredients	pure atelocollagen 3% (of animal origin)
Storage method	Store at room temperature (1-30C) in the shade, do not freeze

